**Medeco BV**  
Alexander Flemingstraat 2  
NL-3261 MA Oud-Beijerland  
The Netherlands


5065	Soft Fine Plus	110 pcs	0.23 mm (32G) x 4 mm
5064	Soft Fine Plus	110 pcs	0.23 mm (32G) x 5 mm
5068	Soft Fine Plus	110 pcs	0.23 mm (32G) x 6 mm
5069	Soft Fine Plus	110 pcs	0.23 mm (32G) x 8 mm
5066	Soft Fine Plus	110 pcs	0.25 mm (31G) x 5 mm
5060	Soft Fine Plus	110 pcs	0.25 mm (31G) x 6 mm
5061	Soft Fine Plus	110 pcs	0.25 mm (31G) x 8 mm
5063	Soft Fine Plus	110 pcs	0.33 mm (29G) x 12 mm

IFU\_KDC011\_06/17-10-2018/03101

# Soft Fine Plus

- Pen Needle
- Pen-Nadeln
- Pennaald
- Pennaald
- Aiguille pour stylo à insulina
- Aguja para pluma de insulina
- Ago per penna
- Penkanyler
- Pennkanyl
- Kanyler for injeksjonspenn
- Kynäneula
- Igła do wstrzykiwaczy
- Pení nöel
- Insulino švirkšto adata
- Pildspalvveida injektora adata
- Tű inzulinoltólhoz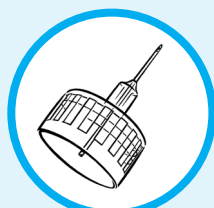

## ENGLISH

### Product Description

- Single use, sterile pen needles for insulin injection.
- Klinion Soft Fine Plus is a pen needle that can be used with most universal insulin pen injectors

### Directions for use

1. Measure the glucose level and determine the insulin dose to administer.  
Remove the seal from the pen needle.
2. Push the pen needle straight on the pen and twist until it is tight. Then pull off the outer protective cap and set it aside. You will need it later to remove the pen needle from the pen.
3. Pull the inner protective cap straight off to prevent damage of the needle.
4. Check the flow by performing an air shot. A drop of liquid should appear at the needle tip.  
Do not use in case the flow is blocked.
5. Set your insulin dose.
6. Clean the injection site (bare skin).  
Make the injection. Do not withdraw the needle immediately after injection.
7. Replace the outer protective cap.
8. Remove the pen needle from the pen and discard immediately.

### Precautions

- Keep out of reach of children without adult supervision.
- Do not re-use. Re-use can cause serious infections. Do not share pen needles. Needles are single use only and should be disposed safely in accordance with your local regulations.
- Do not use if protective seal or tab is damaged or missing, or if the needle appears to be damaged.
- Do not puncture through clothing, since this can damage and clog the needle.
- Consult your healthcare professional for assistance in choosing the appropriate needle length and injection technique.
- Follow your insulin pen manufacturer's instructions for use.
- Remove and dispose of the pen needle immediately after each injection to prevent any damage or contamination, air from entering the cartridge and insulin from leaking out.

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