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PZN Pharmazentralnummer

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EN Instructions for use

The safety pen needle for injection pens is a sterile disposable product. It is used in combination with an injection pen to inject patients with insulin.

Key features

- Automatic needle lock after use, identifiable by the red mark.
- The pen needle is shielded by a needle guard to prevent needle-stick injuries.
- The needle is not visible at any time, which contributes to the patient's comfort.
- The insulin flow through the needle can be verified through the viewing window (preparation test).
- The wing-shaped design ensures a firm and secure grip.
- Available in the two most commonly used needle sizes: 31G 6 mm and 31G 8 mm.
- Suitable for all standard injection pens.

Instructions for use

1. Remove the protective film.
2. Place the safety pen needle straight onto the pen and screw it in place.
3. Remove the outer cap.
Be careful not to put pressure on the needle guard prior to use!
4. Make sure that the needle is visible in the viewing window.
5. Run a preparation test as recommended by the manufacturer of the pen. A drop of liquid should be visible on the tip of the needle through the viewing window.
If the flow test is not successful, use a new safety pen needle.
6. Set the medically prescribed dose of insulin.
7. Insert the needle into the skin in one continuous motion until there is resistance in the needle guard. Inject the dose of insulin according to the doctor's instructions.
8. After the injection has been administered, remove the needle from the skin. The needle guard shields the needle automatically. A red mark will appear in the window. The needle is now locked and can not be reused.
9. Remove the needle from the pen.
10. Dispose of the used safety pen needle in a sharps container.

Warnings

- Keep out of reach of children
- Do not use if the protective film or the outer cap are damaged or missing, or if the needle guard or needle show signs of damage.
- Do not touch the needle guard prior to injection. Applying pressure on the needle guard can trigger the automatic needle lock to prevent needle-stick injuries.
- The safety pen needle cannot be used when the red lock mark appears in the viewing window. This indicates that the needle is locked and cannot be used again.
- Disinfect the area of skin where the injection site will be with an antiseptic agent, in accordance with local standards, before you perform the injection.
- Follow the instructions of the pen manufacturer.

Non-pyrogenic. Non-toxic.
Not made with natural rubber latex.

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