kliniderm[®] alginate standard & alginate cavity

INSTRUCTIONS FOR USE

KLINION

0618 / 1 040404
IFU KW/C018_08

KLINION

The Netherlands
NL-3261 MA Oud-Beljean d
Alexander Flemingstraat 2
Medeco B.V.

ENGLISH

Product Description

Kliniderm[®] Alginate dressings are sterile, nonwoven calcium/sodium alginates consisting of a high G (Guluronic acid) alginate. Kliniderm[®] Alginate dressings are soft, clear gelling, highly absorbent and wet integral allow for intact removal. If used correctly does not stick to the wound. Not made with natural rubber latex.

Indications

Kliniderm[®] Alginate dressings are indicated for moderate to heavily exuding chronic and acute wounds including pressure ulcers, venous leg ulcers, diabetic ulcers, cavity wounds, lacerations, abrasions, post operative surgical wounds and superficial and partial thickness burns. Kliniderm[®] Alginate dressings can also be used to control minor bleeding in superficial wounds.

Contra-indications

Kliniderm[®] Alginate dressings are not indicated to control heavy bleeding or for use as surgical sponge.

Directions for use

Application

1. Select a size of Kliniderm[®] Alginate dressing that is slightly larger than the wound.
2. Cut the dressing to size (using sterile scissors) to prevent overlap of the wound edges. Apply to the wound bed directly.
3. For cavity wounds, loosely fill ensuring that the dressing does not overlap the wound margins.
4. Cover and secure with a semi occlusive secondary dressing, for example Kliniderm[®] Film or Kliniderm[®] Foam dressing.

Dressing Change and Removal

1. Kliniderm[®] Alginate dressings can remain in place for up to 7 days, dependent on patient condition and the level of exudate.
2. Replace Kliniderm[®] Alginate when the dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.
3. To replace, gently remove the dressing. If the wound appears dry, saturate the dressing with sterile saline solution prior to removal.
4. Gently remove the dressing from the wound bed and dispose of according to local procedures and guidelines.
5. If required, irrigate the wound site according to good wound care practice.

Precautions

The dressing may adhere if used on very lightly exuding wounds. Do not use on patients with a known sensitivity to alginates. Re-use can cause serious infections. For external use only. For use under the guidance of a healthcare professional.

References

Size	Pcs	REF
Kliniderm [®] Alginate (for heavy exudate)		
5x5 cm	10	174501
10x10 cm	10	174502
10x20 cm	5	174503
Kliniderm [®] Alginate Cavity (for deep wounds)		
30 cm strips	5	174515

ENGLISH

The following are also available from the range of special wound dressings:

Product name

Kliniderm[®] Film
transparent wound film

Kliniderm[®] Hydro Thin
hydro-active wound dressing –
very thin/transparent

Kliniderm[®] Hydro Standard,
Border, Sacral
hydro-active wound dressing

Kliniderm[®] Foam & Foam border
foam wound dressing

Kliniderm[®] Foam Silicone
silicone foam wound dressing

Kliniderm[®] Alginate Standard &
Cavity
alginate wound dressing

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