kliniderm® foam

INSTRUCTIONS FOR USE

KLINION

ENGLISH

Product Description
Kliniderm® Foam non-adhesive foam dressings are sterile wound dressings, consisting of hydrophilic absorbent polyurethane foam wound contact layer and a pink, low friction waterproof polyurethane film outer layer, which also provides a bacterial barrier. Kliniderm® Foam non-adhesive foam dressing maintains a moist wound environment to assist in the healing process of moderate to heavily exuding wounds.

Indications

Kliniderm® Foam non-adhesive foam dressing may be used throughout the healing process and is indicated for moderate to heavily exuding chronic and acute wounds. Chronic wounds: Pressure ulcers, Venous Leg ulcers, Diabetic ulcers. Acute wounds: Lacerations, Abrasions, Post-operative surgical wounds, Superficial and partial thickness burns and on cavity wounds as a secondary dressing. Kliniderm® Foam non-adhesive foam dressings are suitable for use under compression bandaging. Kliniderm® Foam non-adhesive foam may also be used in the management of infected wounds or wounds in which there is an increased risk of infection under the medical supervision of a healthcare professional. In addition, the dressing aids autolytic debridement when use in combination with a gel, for example the Mesitran Soft.

Directions for use

Site Preparation: Debride when necessary and irrigate the wound site in accordance with standard protocols. Remove excess solution from surrounding skin. Dressing Selection: Select a size of Kliniderm® Foam non-adhesive foam dressing that is slightly larger than the wound.

Dressing Application:

1. If necessary the dressing can be cut using sterile scissors to the wound size.
2. Centre the dressing on the wound with the pink side of the dressing facing up and apply it gently to the wound site.
3. Fix the dressing in place with a suitable tape to ensure the dressing remains in position. Ensure the tape is applied to the edges only and does not cover the dressing as this may affect its performance. Alternatively, if being used under compression bandaging, tape is not required as the bandaging will hold the dressing in situ.
4. Discard any remaining dressing material due to the risk of contamination. Dressing Change and

Removal:

1. Kliniderm® Foam non-adhesive foam dressing can remain in situ for up to 7 days, dependent on patient condition and the level of exudate. Initially it may be necessary to observe the wound or dressing frequently and change the dressing whenever good wound care practice dictates.
2. Gently remove the dressing from the wound bed and discard. Dispose of according to local procedures and guidelines.
3. If required, cleanse the wound site with a suitable wound cleanser prior to application of a new dressing.

Precautions

Ensure the dressing is applied to the wound pink side up. Do not use on patients with a known sensitivity to polyurethane films or foams. Do not re-use in whole or in part, as it may compromise sterility and/or the performance of the dressing. Do not use with oxidising solutions such as hypochlorite or hydrogen peroxide, as these can break down the absorbent polyurethane component of the dressing. Kliniderm® non adhesive foam dressing must not be ingested and must be kept away from children and animals. For use under the guidance of a healthcare professional. For external use only. Does not contain animal products. Do not re-use. Do not re-sterilise. Do not use if individual pouch is damaged/opened. The use by date of this product is printed on the packaging.

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