# kliniderm® hydrocolloid standard - border

## INSTRUCTIONS FOR USE

KLINION®

Medeco B.V. | Alexander Flemingstraat 2 | NL-3261 MA Oud-Beijerland | The Netherlands

KLINION®

## ENGLISH

### Product Description

Kliniderm® Hydrocolloid is a sterile hydroactive selfadhesive wound dressing for moist wound treatment. The dressing maintains a moist wound environment which stimulates the physiological healing process. Kliniderm® Hydrocolloid allows visual assessment of the wound without dressing change. It is very flexible and suitable for the protection of sites exposed to rubbing and shearing stresses. Kliniderm® Hydrocolloid is impermeable to water and bacteria. It does not adhere to the wound, thereby reducing any pain associated with dressing removal preventing trauma to newly formed tissue. For the treatment of chronic and acute wounds with light to moderate exudate.

### Indications

Treatment of low exuding chronic and acute wounds. Superficial chronic wounds (pressure sores, leg ulcers). Superficial acute wounds (burns, abrasions, donor sites). Post operative wounds. Prevention of Pressure sores.

### Contra-indications

Do not use on: Ulcers caused by chronic infections (tuberculosis, deep mycotic infections, syphilis ...). Arterioopathy stage IV. Bites and 3rd degree burns. Wounds showing clinical signs of infection (temperature, pus, inflammatory signs) should be treated under medical supervision before use of kliniderm® Hydro, can be resumed.

### Directions for use

**Skin Preparation:**  
Clean the wound using sterile saline or Ringer's solution and sterile swabs. Dry the wound edges. Remove any remains of creams or greasy substances.  
**Dressing Application:**  
Choose an appropriate size - the dressing must extend 2-3 cm beyond the edges of the wound. Remove the printed protective paper. Apply the adhesive side of the dressing onto the wound and gently press on the edges of the dressing. No additional fixing is necessary.  
**Dressing Changes:**  
Kliniderm® Hydrocolloid can be left on the wound for up to 7 days. The dressing should be changed if a gel bubble has developed and has attained the same size as the wound itself. Dressings on post operative wounds should be changed when the sutures are to be removed. Remove debris and gel from the wound using swabs. Clean the wound as described above before applying a new dressing.

### Precautions

Special notes: Kliniderm® Hydrocolloid should be left in place as long as possible in order to prevent trauma to the fragile newly formed tissue and to reduce cross contamination through frequent dressing changes. A yellowish gel with characteristic odour can develop under the dressing. This is not a sign of infection but a normal phenomenon when using hydrocolloid dressings which will disappear after cleaning the wound. Kliniderm® Hydrocolloid maintains a moist wound environment which is conducive to the natural healing process. Kliniderm® Hydrocolloid can be used in conjunction with compression therapy. It may also be used as a protective dressing on 'at-risk', undamaged skin or on skin beginning to show signs of damage from friction or shear. Treatment of any skin ulcer should be part of a well-defined plan for ulcer management and under the supervision of a health care professional. Re-use can cause serious infections.

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