
# klinidur textelast®

INSTRUCTIONS FOR USE

**KLINION®**

IFU\_KWC050\_02

Medeco B.V. | Alexander Flemingstraat 2  
NL-3261 MA Oud-Beijerland | The Netherlands

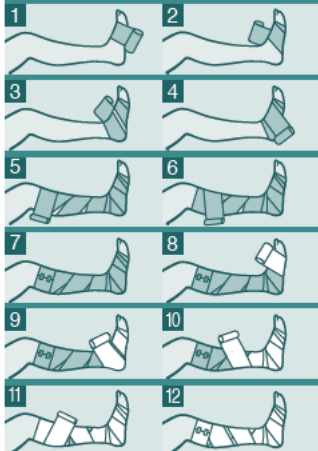## ENGLISH

Ambulant compression therapy using the Klinidur Textelast® short stretch bandage

### Aim

To reduce and/or prevent oedema together with venous ulcer. Ambulant compression therapy provides high working pressure and low resting pressure on the lower leg. Consequently the bandage can remain in place overnight. It is vitally important for this treatment that the patient moves frequently.

### Method

The compression therapy showed in the pictures requires two short stretch bandages that are applied to the lower leg in opposite directions. You can use one bandage, depending on your local protocol.

### Important

Whichever technique you use, you should bear in mind the following: the foot should be kept straight and at right angles with the leg when applying the bandage. Whilst bandaging, ensure that the bandage remains stretched and keep the roll short. Always bandage around the heel; the stretch bandage should have no openings over the heel. It is recommended that a dressing be placed around the lower leg prior to bandaging. Ensure that the leg is sufficiently padded to prevent blisters, weals and pressure points. The bandage should always be secured with adhesive plaster. Do not use directly on open wounds. Do not use if skin is damaged.

### Instructions

1. Start with the first Klinidur Textelast® short stretch bandage at the big toe and bring this over the top of the foot towards the little toe.
2. Then bring the bandage once over the middle of the foot thereby securing the bandage.
3. Now bandage towards the middle of the heel whereby the bandage is turned for two-thirds around the side of the heel and one-third underneath it.
4. Then bandage the still exposed part of the foot with one single turn.
5. Next, apply the bandage in one turn horizontally above the ankle and then carry on bandaging around the leg to just under the knee.
6. Just under the kneecap make one circular turn with the bandage and then bandage down the leg covering any exposed parts.
7. Finally, bandage upwards again. Secure the bandage with two pieces of adhesive plaster.
8. Steps 8 to 12 (only if you use the 2 bandage technique) Repeat steps 1 to 7 with the second Klinidur® short stretch bandage but in the opposite direction. Start bandaging at the little toe and work in the direction of the big toe.

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